


ULTRA SAFETY PLUS

TWIST XL

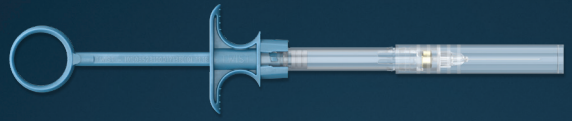

DO's & DON'Ts




Do not use the Ultra Safety Plus black handle



Use the blue Ultra Safety Plus Twist™ handle

Do not twist the needle cap upon removal

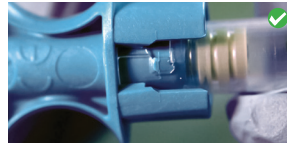

Lock the handle and the syringe together by twisting. Ensure that they stay locked throughout the whole procedure.

Do not apply excessive pressure during the procedure



Before the injection, remove the needle cap by pulling it straight apart



Towards the end of the cartridge, if you feel any resistance, stop the injection



Fully retract the plunger before separating the device

Do not bend the two components to separate



Rotate the barrel to **unlock the system** and carefully pull the two parts straight apart to separate

