FR

COMPOSITION

Penghawar djambi, eugenol, sodium lauryl sulphate, calcium carbonate, arôme menthe, excipients.

PROPRIÉTÉS

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PROPERTIES

ALVEOXYL is a paste for alveolar surgical dressing after tooth extractions, with Penghawar fibers. The paste promotes haemostasis by compression and protects from superinfection (alveolar osteitis) by its barrier effect.

Introduced into the socket, ALVEOXYL easily adheres to the alveolus due to its fibrous consistency. Assisted by the patient's tongue movements, it gradually self eliminates without an intervention by the practitioner as the normal healing of the wound takes place.

ALVEOXYL is very easy to apply and requires no special attention from the practitioner other than observation of the healing process.

INDICATIONS

ALVEOXYL is a paste used as dressing in case of dry socket or post-extraction dressing following a difficult or traumatic extraction in patients with history of dry sockets.

CONTRA-INDICATIONS

ALVEOXYL is not to be used on patients having a history allergy to one of the components and on deciduous teeth (children under 12 years of age).

Routine use of the product as a post-extraction dressing is contraindicated.

INSTRUCTIONS FOR USE

Take a small pellet (about 0.20 g) of paste and place gently it into the prepared dental socket.

Do not suture.

WARNINGS AND PRECAUTIONS OF USE

Patients should be told not to wash their mouth vigorously within the 24 hours following the avulsion.

Do not treat more than 5 teeth in one session.

Warning

Contains eugenol and mint flavour ((S)-p-mentha-1,8-diene) Causes serious eye irritation.

May cause an allergic skin reaction.

Harmful to aquatic life with long lasting effects.

Refer to safety data sheet.

The cap should always be replaced immediately after use.

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